Sparrow Laboratories SCLM



Urine

Freshly voided, clean-catch urine, catheterized urine, ileal conduit urine, cystoscopy urine, bladder washing, urethra, ureteral, renal pelvis

Each specimen container needs to have a *minimum* of two (2) patient identifiers to be accepted.

- Label container with patient's full name (first and last), date of birth or medical record number (MRN). Please indicate specimen type on container. If appropriate, specify left or right (ureteral and renal pelvis washings).
- Place fresh urine in clean, leak-proof container.
- An accompanying requisition with patient's full name (first and last name), date of birth, medical record number (MRN), specimen source, ordering physician's name, date of collection, diagnosis, and pertinent patient history.
- Refrigerate fresh specimen until transport to lab.
- CytoLyt fixative can be added to specimen that will be delayed.