

LAB	CODE	TEST NAME
Evergreen	Q FEVE-LC	Q Fever AB, IgG
Evergreen	QTGLDINC-LC	Quantiferon TB Gold- Micro Incubate
LABCORP	199995	QAPAM
LABCORP	182879	<p>QuantiFERON(R)-TB Gold Plus</p> <p>Container:The QuantiFERON(R)-TB Gold Plus collection kit contains the instructions for the collection and handling of (one each): (1) gray-top (with white ring), uncoated (nil); (2) green cap with white ring, TB1 tube; (3) yellow cap with white ring, TB2 tube; (4) purple top with white ring, mitogen-coated. A high altitude kit is also available for locations between 3350 and 6150 feet; a cap with a yellow ring differentiates the tubes.</p> <p>Temperature:Maintain specimen at room temperature for four tube collection kit. Stability: Kit MUST be incubated within 16 hours of collection.</p>
LABCORP	182913	<p>QuantiFERON-TB Gold Plus</p> <p>Container:Green-Top (Lithium Heparin) Tube</p> <p>Temperature:After collection, the blood can be held at room temperature (17 to 25 degrees Celsius) for up to three hours, and then placed at 2 to 8 degrees Celsius for up to 48 hours. Alternatively, blood can immediately be placed at 2 to 8 degrees Celsius for up to 48 hours. *After refrigeration, the lithium heparin tube MUST equilibrate to room temperature (17 to 25 degrees Celsius) prior to the transfer to QFT-Plus Blood Collection Tubes. *Aliquoted QFT-Plus Blood Collection Tubes should be placed in the 37 degree Celsius incubator within two hours of blood transfer. *NOTE: Total time from blood draw to incubation in QFT-Plus Blood Collection Tubes should NOT exceed 53 hours.</p>
LABCORP	808404	<p>Quetiapine, Serum or Plasma</p> <p>Container:Red-top tube OR green-top (heparin) tube. Gel-barrier tubes are NOT recommended.</p> <p>Temperature:SUBMISSION/TRANSPORT (&lt;3 days): Room Temperature. For storage beyond three days, specimen should be refrigerated or frozen.</p>
LABCORP	791462	<p>Quetiapine, Ur</p> <p>Container:Urine transfer tube</p> <p>Temperature:Maintain specimen at room temperature up to three days. Specimens may also be refrigerated or frozen.</p>
LABCORP	007831	<p>Quinidine, Serum</p> <p>Container:Red-top tube OR green-top (heparin) tube. DO NOT USE A GEL-BARRIER TUBE. The use of gel-barrier tubes is not recommended due to slow absorption of the drug by the gel. Depending on the specimen volume and storage time, the decrease in drug level due to absorption may be clinically significant.</p> <p>Temperature:Room Temperature</p>