

Online Test Menu Laboratory Catalog



LAB	CODE	TEST NAME	CPT4 CODE
SJMC	LAB96	Lactate Dehydrogenase Container:GREEN-LI HEP, alt: GOLD-SST or RED TOP-PLAIN Temperature:Refrigerated Place Of Service:Providence St Joseph Medical Center Laboratory	83615
SJMC	LAB2377	LACTATE DEHYDROGENASE, BODY FLUID Place Of Service:St. Patrick Hospital	83615
SJMC	LAB95	Lactic Acid Container:GRAY-K OXALATE/NA FLUORIDE Temperature:Refrigerated Place Of Service:Providence St Joseph Medical Center Laboratory	83605
SJMC	LAB99	Lipase Container:GREEN-LI HEP, alt: GOLD-SST or RED TOP-PLAIN Temperature:Refrigerated Place Of Service:Providence St Joseph Medical Center Laboratory	83690
SJMC	LAB13233	LIPASE, BODY FLUID Place Of Service:St. Patrick Hospital	83690
SJMC	LAB2478	Lipid Panel Container:GREEN-LI HEP, alt: GOLD-SST or RED TOP-PLAIN Temperature:Refrigerated Place Of Service:Providence St Joseph Medical Center Laboratory	80061
SJMC	LAB29	LITHIUM LEVEL Place Of Service:St. Patrick Hospital	80178
LABCORP	007012	Lacosamide Container:Red-top tube, lavender-top (EDTA) tube, OR green-top (heparin) tube. DO NOT USE A GEL-BARRIER TUBE. The use of gel-barrier tubes is not recommended due to slow absorption of the drug by the gel. Depending on the specimen volume and storage time, the decrease in drug level due to absorption may be clinically significant. Temperature:Maintain specimen at room temperature.	80235
LABCORP	620044	Lactate (CSF) Container:Sterile screw capped vial Temperature:Freeze as soon as possible after collection	83605
LABCORP	023804	Lactate and Pyruvate Container:2 Gray-top (sodium fluoride) tube Temperature:Refrigerate	84210 83605
LABCORP	001842	Lactate Dehydrogenase (LD) Isoenzymes Container:Red-top tube or gel-barrier tube Temperature:Maintain specimen at room temperature. Do NOT freeze. Do NOT refrigerate for more than three days.	83615 83625
LABCORP	100156	Lactic Acid Dehydrogenase (LD), Body Fluid Container:Sterile body fluid container Temperature:Room temperature	83615
LABCORP	812030	Lactic Acid, Body Fluid Container:Sterile container Temperature:FREEZE	83605
LABCORP	122408	Lactic Acid, CSF Container:Clean sterile tube Temperature:Refrigerate for up to 15 days or freeze. Stable at room temperature up to 3 days.	83605
LABCORP	120543	Lactic/Pyruvic Ratio Container:Gray-Top (Sodium Fluoride) Tube	84210 83605