

LABORATORY CATALOG

Where good health begins.

LAB	CODE	TEST NAME	CPT4 CODE
LRH	QUANTBP	QUANTIFERON TB GOLD PLUS	
LRH	SEROQU	QUETIAPINE (SEROQUEL)	
LRH	QUIN	QUINIDINE	
LABCORP	016774	Q Fever Antibodies, IgG Container:Red-top tube or gel-barrier tube Temperature:Maintain specimen at room temperature.	86638
LABCORP	806617	Q Fever IgM Antibody Screen Container:Red-top tube or gel-barrier tube Temperature:Refrigerate	86638 x2
LABCORP	199995	QAPAM	NOCPT
LABCORP	182913	QuantiFERON-TB Gold Plus Container:Green-Top (Lithium Heparin) Tube Temperature:After collection, the blood can be held at room temperature (17 to 25 degrees Celsius) for up to three hours, and then placed at 2 to 8 degrees Celsius for up to 48 hours. Alternatively, blood can immediately be placed at 2 to 8 degrees Celsius for up to 48 hours. *After refrigeration, the lithium heparin tube MUST equilibrate to room temperature (17 to 25 degrees Celsius) prior to the transfer to QFT-Plus Blood Collection Tubes. *Aliquoted QFT-Plus Blood Collection Tubes should be placed in the 37 degree Celsius incubator within two hours of blood transfer. *NOTE: Total time from blood draw to incubation in QFT-Plus Blood Collection Tubes should NOT exceed 53 hours.	86480
LABCORP	791462	Quetiapine, Ur Container:Urine transfer tube Temperature:Maintain specimen at room temperature up to three days. Specimens may also be refrigerated or frozen.	80342
LABCORP	810819	Quinine, Serum/Plasma Container:Red-top tube or lavender-top (EDTA) tube Temperature:Refrigerate	84228
LABCORP	802110	Quinine/Quinidine Different. Container:Red-top tube, lavender-top (EDTA) tube Temperature:Refrigerate (Cool Pack)	80194
LABCORP	805913	Quinoa (Chenopodium quinoa)IgE Container:Gel-barrier tube Temperature:Maintain specimen at room temperature	86003
LABCORP	827488	Quinoline Yellow - BasoFu. HRT Container:Green-top (sodium heparin) tube: submit original tube Temperature:Maintain specimen at room temperature	86343